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Ocu+™ Sodium Hyaluronate (NaHA) Sterile Ophthalmic Viscosurgical Device for Intraocular Use... Description: Ocu+™ is a clear, sterile, non-pyrogenic cohesive viscoelastic preparation of high molecular weight sodium hyaluronate...

Table with 2 columns: Viscositeit (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

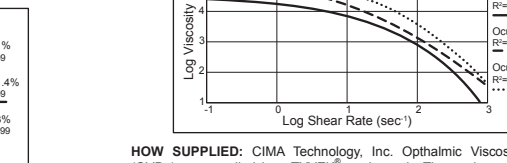
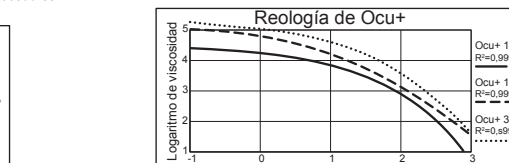
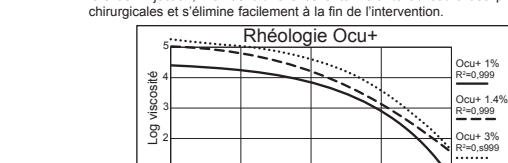
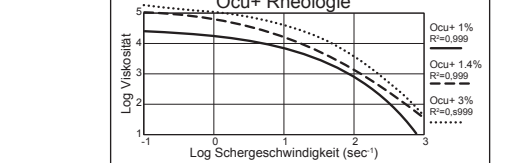
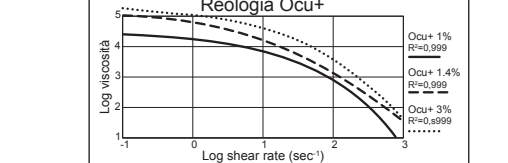
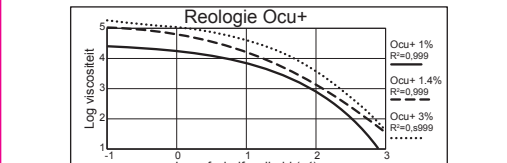
Table with 2 columns: Viscosità (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

Table with 2 columns: Viskosität (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

Table with 2 columns: Viscosité (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

Table with 2 columns: Viscosidad (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

Table with 2 columns: Viscosity (cps) and NaHA/ml. Rows show viscosity ranges for different concentrations (1.0%, 1.4%, 1.6%, 1.8%, 2.0%, 3.0%) at 10, 14, 18, 20, and 30 mg/ml.

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HOW SUPPLIED: CIMA Technology, Inc. Ophthalmic Viscosurgical Devices (OVDs) are supplied in a TYVEK™ peel pouch... EXPIRATION DATE: The expiration date on the package is the sterile expiration date... CONTRAINDICATIONS: At present, there are no known contraindications when used as recommended...

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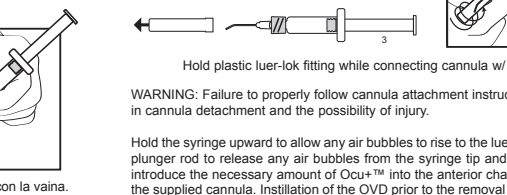
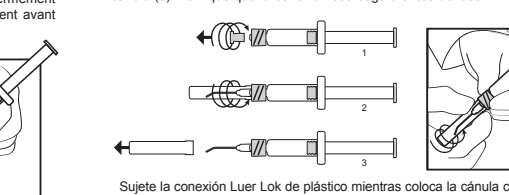
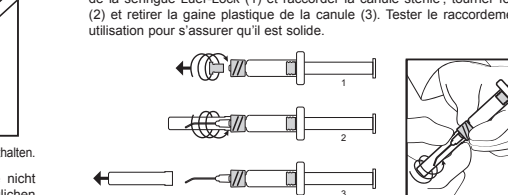
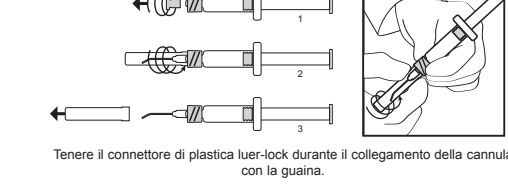
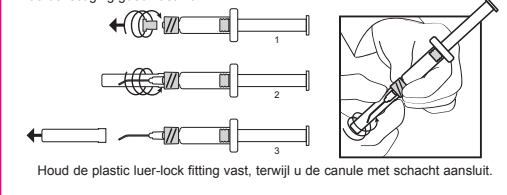
INDICATIONS: Ocu+™ is to be used as an ophthalmic surgical aid during anterior segment surgery... INSTRUCTIONS FOR USE: Before use, allow the solution to attain room temperature for about 20 minutes... PRECAUTIONS: Precautions associated with the use of Ocu+™ relate to the surgical procedure being performed...

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WARNING: Failure to properly follow cannula attachment instructions may result in cannula detachment and the possibility of injury... Hold the syringe upward to allow any air bubbles to rise to the luer tip...

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PRECAUTIONS: Precautions associated with the use of Ocu+™ relate to the surgical procedure being performed... 1. Do not overfill the anterior chamber with the supplied OVD.

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ADVERSE REACTIONS: During ophthalmic surgical procedures, OVDs are well tolerated after injection... 1. Do not use Ocu+™ in the presence of active inflammation.

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REPORTING: Adverse events and/or potentially sight threatening complications should be promptly reported as they are related to the solution... 1. Report any adverse events or potentially sight-threatening complications...

Ocu+™ Natriumhyaluronat (NaHA) Sterile. OP DE VERPAKKING GEBRUIKTE SYMBOLEN. Symbol description table for Ocu+™.

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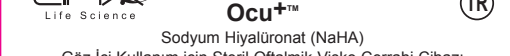
Table with 2 columns: Viskozite (cps) and NaHA/ml. Shows viscosity values for different Ocu+ concentrations (1.0% to 3.0%).

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TEDARİK YERLERİ: CIMA Technology, Inc. Ofislerindeki Sterilizasyon (OVD) bir TYVEK® soyulmuş kesir...