

Specialists in Paediatric Vision Testing



About us

For 40 years we have used our industry expertise to provide the best and most reliable orthoptic products, so eye professionals can test and treat children’s vision with confidence.

The Gold Standard picture vision test for pre-literate children

Our business was founded in 1984 by orthoptist Hazel Kay. Hazel researched and developed a new paediatric vision testing system using recognisable optotype pictures, to enable earlier visual acuity measurements of very young children and those with learning disabilities.

The Kay Picture Test is the only fully researched and validated picture visual acuity test that provides consistent, subjective visual acuity earlier than letter and symbol tests.

Early testing and intervention will prevent avoidable vision loss

We make the difference by providing tests to engage babies and children in a professional and

fun way. Successful eye examinations rely on cooperation from the child, so our scientifically and clinically validated vision tests and accessories have been developed to make testing quick and easy.

Using our age-appropriate vision tests will ensure that the child’s interest, attention, and understanding are maintained throughout testing.

Kay Pictures’ product range is used by over 100 NHS Trusts in the UK as well as eye care professionals, school nurses, health visitors, teachers of the visually impaired and paediatricians worldwide.



Shipping charges apply to all orders. There is a minimum shipping charge of £6.50 for UK orders. When purchasing online, the shipping costs are displayed at checkout. International customers are advised to check with their local customs authority prior to placing an order whether there will be any additional duties or charges payable.

For more details, please visit: www.kaypictures.com/buying-from-us

How to order

All products listed in this brochure can be purchased online at www.kaypictures.com

NHS and State Education official purchase orders are automatically accepted for account invoicing.

We also accept written orders by email - please see our contact details below. Orders from new and international customers will need to be paid in advance. Existing customers who order regularly will be set up with a 30-day account.

Contact us

Kay Pictures Ltd
Unit 39, Silk Mill Business Park
Brook Street, Tring, Hertfordshire HP23 5EF
T +44 (0) 1442 823507
E contact@kaypictures.com

Connect with Hazel

Sign up to our newsletter



Our research

The Kay Picture Test is the only fully researched and validated picture visual acuity test that provides consistent, subjective visual acuity earlier than letter and symbol tests.

In a comprehensive research study carried out by Liverpool University Directorate of Orthoptics and Vision Science and a number of clinical orthoptic departments across the UK and The Retina Foundation of the Southwest, USA, the Kay Picture Test was shown to be valid and repeatable within the limits of current gold standards with high testability.^{1,2}

Recent normative data collected for the redesigned Kay Picture Visual Acuity Test shows that those tested with the Kay Picture Test, in comparison to previously published norms of HOTV and Lea Symbols, have more consistent mean visual acuity scores across all ages and less variability in acuities (numerically lower standard deviations).³

1. O'Connor AR Kay H, Thomson D, Newsham D. Redesigning the Kay Picture Test for Children. Invest Ophthalmol Vis Sci April 2010 Volume 51, Issue 13.
2. <http://doi.org/10.22599/bioj.97>
3. O'Connor AR, Kay H, Milling A. Normative data for the redesigned Kay Picture visual acuity test Jaapos 2020.05.003



Table of Contents

Vision Testing4

Apps.....10

Contrast Sensitivity13

Occluders 14

Orthoptics Tests 16

Occlusion Therapy 19


Fixation Tools & Accessories23

Reward Stickers..... 26

Free Posters28



Click here for an overview of our Kay Picture Tests

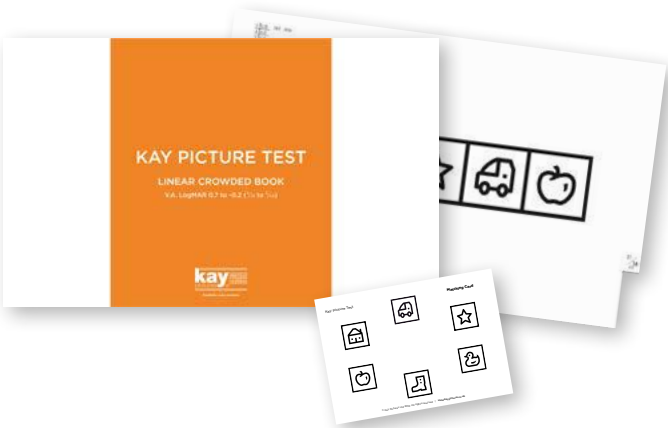

Kay Picture Test Linear Crowded Book

Linear picture acuity test with distance LogMAR sizing at three metres from 0.70 to -0.20, plus near testing and matching card. The pictures are presented in a row of five boxes to provide consistent crowding. There are four different arrangements of the picture optotypes at each acuity size. Suitable for naming and matching. Age 30 months+.

Order code: KPTLINEAR

Kay Picture Test Single Crowded Book

Single Crowded picture acuity test with comprehensive distance LogMAR sizing at three metres from 1.00 to -0.10, plus near testing and matching card. Six picture optotypes are presented in a crowding box with a choice of four different optotypes at each acuity level. Suitable for naming and matching. Age 18 months+.

Order code: KPTSINGLE



Kay Picture Test Matching Card



Double-sided matching card for use with all picture vision test books and charts plus our iSight Test Pro app and licensed digital vision testing software.

Order code: KPTKC

Kay Picture Test Screening Chart

Double-sided distance chart for vision testing at school entry, GP practice, or other screening environment without the child needing to know capital letters. Very quick and easy test process with single and line presentation. Age 4+.


Order code: KPTCHART3M

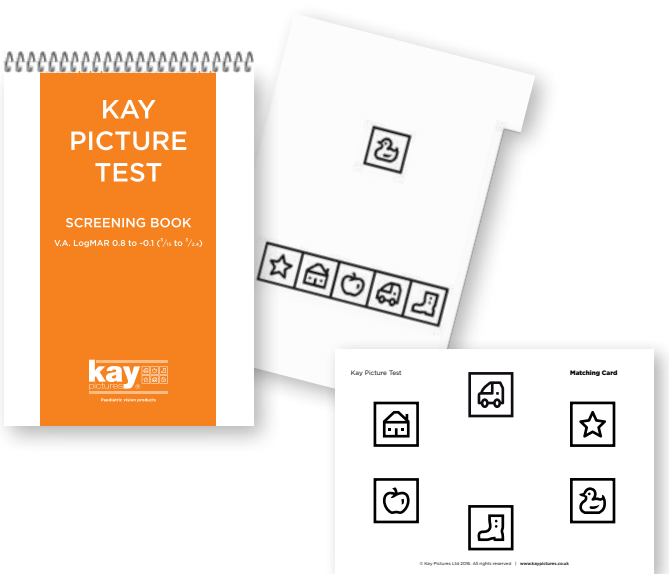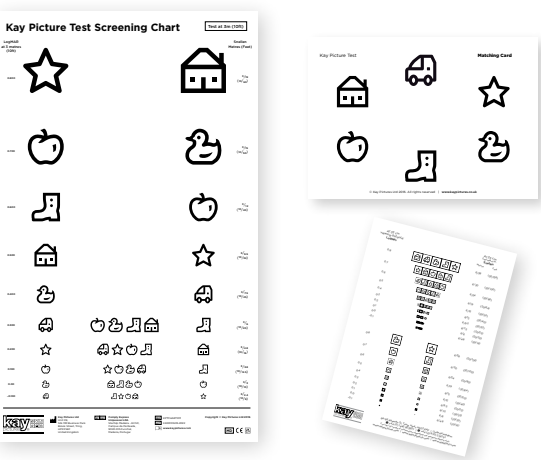

Kay Picture Test Screening Book

Compact A5 size book containing an abridged single crowded and linear crowded picture vision testing option for vision screening. Comprises distance LogMAR sizing from 0.80 to -0.10 at three metres, plus matching card. Suitable for naming and matching. Age 18 months+.

Order code: KPTSCREEN

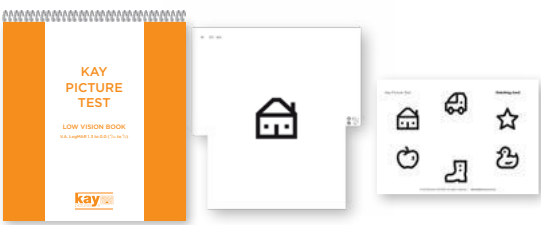

Kay Picture Test Low Vision Book

The Low Vision Test is specifically targeted at testing those with reduced vision. It has single, uncrowded picture presentation in fourteen LogMAR sizes, which at three metres is from 1.3 to 0.0, plus a near vision test and matching card. Age 18 months+.

Order code: KPTLOW

Recognition Booklet

Our DIY Kay Picture Test recognition booklet provides a fun way for a child to learn the Kay Picture optotypes prior to testing to improve their familiarity and confidence with the pictures. The booklets are made from a single A4 sheet which is folded into a six-page book showing each optotype and a coloured drawing. Making and colouring the booklet is all part of the fun! Pack of 100 sheets.

Order code: KPTR100




Kay Picture Test Near Vision Test Cards

Laminated cards for near vision testing, as found in our near and distance book sets.

Kay Picture Test Crowded Near Test Card

Comprises crowded single and linear pictures for testing near vision at 33cm distance. There is a different arrangement of pictures on each side of the card to test each eye. Matt laminated. A5 double-sided.

Order code: KPTN4

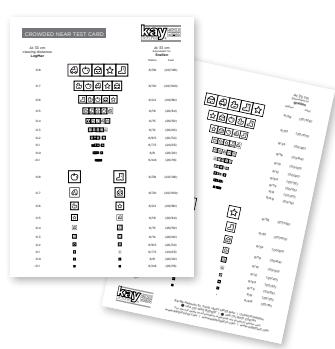

Kay Picture Test Uncrowded Near Test Card

Comprises uncrowded pictures from 1.3 to 0.0 LogMAR (6/120 to 6/6 Snellen equivalent) at 33cm testing distance. Matt laminated. A4 double-sided.


Order code: KPTN5



Key Letter Test

Linear letter acuity test comprising six of the Sloan letters used in the ETDRS study and regarded as the International Standard. It contains comprehensive LogMAR sizing at three metres from sizes 1.00 to -0.20, plus near testing with LogMAR letters and high frequency words. Snellen equivalent sizes in metres and feet are also given.

Order code: **KLTND**

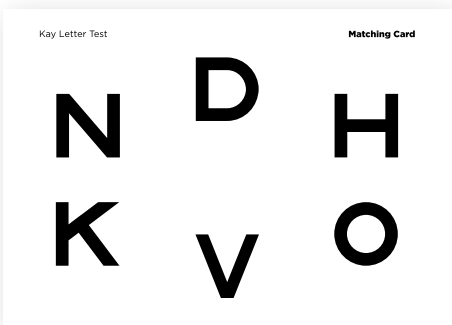

Key Letter Test Matching Card


For use with the Kay Letter Test and iSight Pro app. This matching card is A5-sized and fully encapsulated. An uncrowded arrangement of the six Sloan letters is printed on one side. We recommend it is shown to the child before testing to give them confidence in naming the letters or to practice matching, if required.

Order code: **KLTMC**

Key Letter Test Near Test Card

Comprises reduced size LogMAR letters from 1.0 to -0.1 (6/60 to 6/4.75) at 33 cm, and high frequency words text from N48 to N5 sizes for young readers. Matt laminated. A5, double-sided.

Order code: **KPTN1**

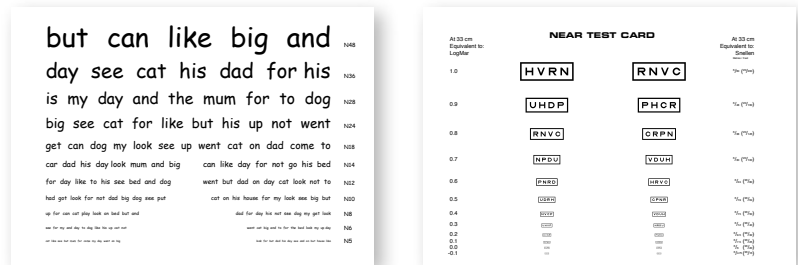

Child Trial Frames

Comfortable and light weight frames with adjustable temples. The frames hold three trial lenses in the front and one lens behind each side. Knobs located on the front allow for easy axis rotation to define the astigmatism axis. The frames come in a protective case.

Order codes:

Infant, Interpupillary distance 56mm, purple frame: **TRF56**

Child, Interpupillary distance 58mm, red frame: **TRF58**



3 Metre Tape Measures

Ideal for measuring correct vision testing distances as well as many other test measurements. Made from soft, strong PVC-coated fibreglass with metal ends. It is 1.7cm wide, safe to handle and washable. Double-sided, showing either centimetres or inches with a black and white colour scheme which makes them easy to read.

Order codes:

Pack of 5: **TAPE5**

Pack of 10: **TAPE10**

Kay Acrylic LogMAR Vision Screening Charts with optional LED Illuminated Box

LED Illuminated Box

Lightweight LED illuminated box holding a single vision testing acrylic panel with spotlight below the panel. Manual side switch to select illuminated sections. 12V medical power plug and 2m lead. It is white, can be wall-hung or stood on its feet. The LED strips should last for 30-50k hours. Choose optional carry handle or wall bracket.

Large box with carry handle: 52.5cm x 21cm x 5.2cm

Large box with wall bracket: 52.5cm x 21cm x 18cm

Compatible Kay Charts:

KE3MPIC
KE3MLET- HN
KE3MLET- KH
KE3MLET- KHMIRR

Small box with carry handle: 37.7cm x 15.7cm x 5.2cm

Small box with wall bracket: 37.7cm x 15.7cm x 18cm

Compatible Kay Chart:


KES3MLET-VHK



Stability feet



Box with Wall Bracket

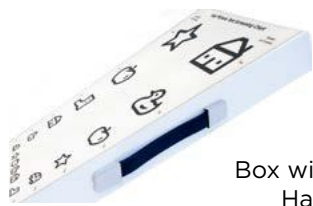

Box with Carry Handle



Kay Picture Test 3 Metre Acrylic LogMAR Chart

Translucent LogMAR picture chart measuring visual acuity from 0.80 to -0.10 (Snellen 3/19 – 3/2.4). Two vertical columns allow each eye to be tested with a different selection of optotypes. The central row of four pictures can be scored to provide a crowded acuity measurement per optotype.

Chart dimensions: 47.8cm x 20.4cm x 0.2cm


Order codes:

Chart: **KE3MPIC**

Chart & Light Box: **EVBOX1-PIC**

Chart & Light Box with Handle: **EVBOX1H-PIC**

Chart & Light Box with Wall Bracket: **EVBOX1WB-PIC**

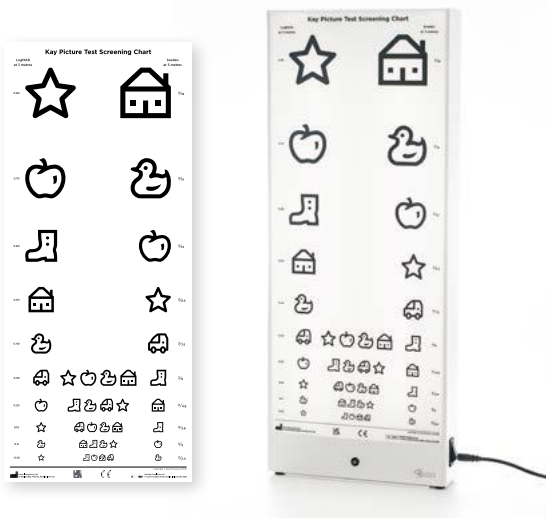

Single Acrylic Chart

Chart in Box

Kay iSight Test Professional

The Kay iSight Test Professional app lets you measure and record near and distant visual acuity in children from 18 months of age through to adults. It is simple and intuitive to use, making your testing quick and easy.

Test results can be safely stored on your device, exported to a spreadsheet or deleted, as required.


The app uses international standard letters and Kay Picture optotypes for young children and others with a learning disability. In addition, near reading text is provided as short, paragraphs or single, high frequency words. There is also a point of light fixation target plus 4 animated images and two static scenes.

The app can be used to test vision remotely by screensharing through a stand-alone video conferencing app at your desk to test anywhere in the world.


Using a suitable Bluetooth mouse, the app can be operated remotely to test at distance.

This app is registered and CE marked as a medical device with the MHRA.

Shop online at: www.kaypictures.com

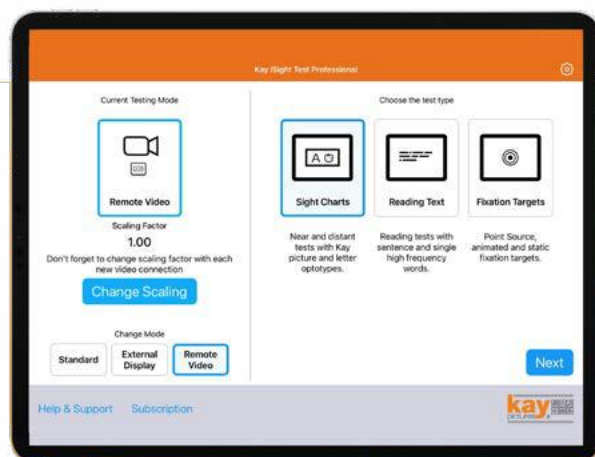

Watch a short summary of the app's main features here

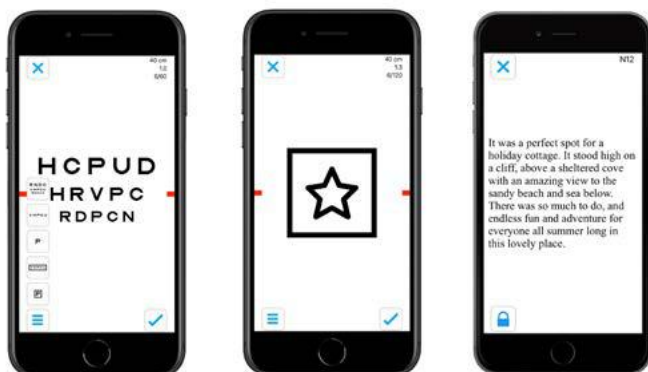

Available on the
App Store

NEW
UPDATE



Features:

- Share (iPad only) to another device for remote vision testing
- Various testing distances from 33cm to 6 metres
- Choice of letters or pictures for testing a range of abilities
- Displays LogMAR notation, metric, imperial Snellen, or Decimal
- Records and saves visual acuity results for each eye
- Export or delete saved test results
- Offers various single, crowded and chart presentations
- Reading text in 12 sizes from N48 to N5
- Sentences of exactly 50 words to facilitate reading speed testing
- Lock screen on reading pages
- Easy to increase and decrease size displayed
- Refresh optotypes or text on screen
- Coloured, animated fixation targets to maintain interest and accommodation

The FREE setting provides a distance letter test and new subscribers have a one-month free trial to test all the additional features. You can cancel the subscription at any time during the introductory free period and pay nothing. If you wish to use the app after that, payment will be taken by iTunes.

Kay iSight Test Business

Kay iSight Test Business provides access to the same content as Kay iSight Test Professional specifically for organisations using **Apple Business / Schools Manager and MDM distribution**.

There will be a small, initial download cost per device to access the app as well as a fixed-term, annual licence fee paid directly to Kay Pictures Ltd which ensures the app installs updates and continues to function on any new Apple device and iOS update during the term of the agreement.

Please get in touch for a quote and to obtain licences.

[Find out more](#)



GET IT ON
Google Play

Available on the
App Store

Kay Say & Match

The FREE* interactive app for familiarisation with the Kay Picture optotypes.

The Kay Say and Match app provides a fun way to help very young children to practise and learn the Kay Picture optotypes prior to having an eye test at home.

There are four games in the Kay Say and Match app, three of which are free to download and use on an iPad or iPhone. The fourth game is a talking matching card which is an in-app purchase.



The **Name** game teaches the names and sounds of the six Kay Picture optotypes. Knowing the pictures and names will give the child confidence to perform a professional vision test earlier than otherwise.



The **Match** games help children to learn the concept of matching the same picture, which may be useful during a vision test if the child is shy or non-verbal.



The **Practice** game simulates the key aspects of a vision test, where the picture to be matched is shown separately, remembered, then matched.



The **Test** game is an in-app purchase and provides a speaking matching card to be used during a professional vision test. This is particularly helpful for children with additional needs, such as autism.

* Includes in-app purchases.



Kay Amblyopia Tracker

The Kay Amblyopia Tracker allows eye professionals to give their patients a way to monitor changes in their child's vision at home between clinic appointments.

The app provides an easy, automated test process for parents to follow. Results are stored safely only on their phone and displayed in a simple chart that can be shared with their eye professional by email. The app's results chart gives feedback on the positive effect of amblyopia treatment, vision monitoring or during refractive adaptation.

Vision is measured with accurate letter or picture optotypes, but with a different method from a clinical test, thus ensuring that parents understand the role of the app is monitoring changes, not replacing their clinical care.

Regular home testing aims to motivate patients by encouraging active involvement in their treatment. It can reassure concerned parents, alert to possible issues between appointments, or even reduce the frequency of some follow-up clinical appointments.

The app is free to download on both Android and iOS smartphones, then is activated by a code provided by the eye professional, ensuring that home testing is part of ongoing eye care.

The Kay Amblyopia Tracker has been through a full NHS ethical research project and has a peer-reviewed study published that validates the testing method. [Available here.](#)




FREE TRIAL

Are you an eye professional interested in offering our Amblyopia Tracker app to your patients?
A FREE TRIAL is now available!

[Get in touch](#)

to receive further information.

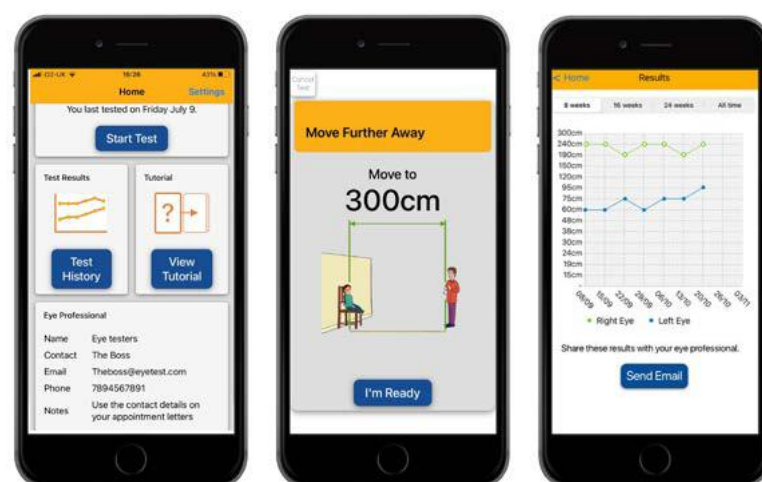

Mayer-Kran™ Double-Happy™ Low Contrast Test

Cards with a schematic smiling face offset from the centre and identical when rotated 180°. Contrasts range from nearly black (89%) to very dim (2.2%). The face contrast values are converted into log contrast sensitivity.

The Screening Test includes 6 cards (0.05 -1.65 log CS) and the full test has 16 cards (0.05 - 2.1 log CS, one blank card).

Order codes: Screening Test: **PV2012-6**

Full Test: **PV2012-16**

SpotChecks™

Single use tests for patients for self-testing by marking each contrast target they see. Testing is intuitive, only takes 2-3 minutes, and scoring the results is very easy. The test sheet becomes a record for the patient file. The Introductory Set includes 20 sheets (5 sets of tests A-D, 1 recording form) and the Full Set has 120 sheets (10 sets of tests A-L, 12 recording forms).

Order codes: Introductory Set: **PV5031-KitA-D**

Full Set: **PV5031-KitA-L**



Pelli-Robson Traditional Contrast Sensitivity Charts

Pair or individual charts for use at 1m testing distance with Sloan letter optotypes. Letter sequences are organised in groups of three with two triplets per line. Within each triplet, all letters have the same contrast. The contrast decreases from one triplet to the next. May be wall or easel mounted.

Order codes: Charts 1 & 2 (HSZ and VRS): **PV5014**

Chart 3 (SDN): **PV5014-3**

Chart 4 (NSD): **PV5014-4**



NEW

Pelli-Robson Trans-Illuminated Contrast Sensitivity Charts

These Sloan letter charts cover $\frac{3}{4}$ of the original contrast range. The Low Vision Charts use the upper 12 contrast levels (0 - 1.65 Log CS). The charts for Peak Contrast Sensitivity use the lower 12 contrast levels (0.6 - 2.25 Log CS). The A and B versions show different letter sets to minimise memorisation.

Trans-illumination has the advantage of uniform fixed illumination. Suitable for the ETDRS Illuminator Cabinet™ (63cm x 65.5cm x 17.8cm).

Order codes:

Low Vision

Full Set (A & B): **PV5016**

Chart A: **PV5016A**


Chart B: **PV5016B**

Peak Vision

Full Set (A & B): **PV5017**

Chart A: **PV5017A**

Chart B: **PV5017B**

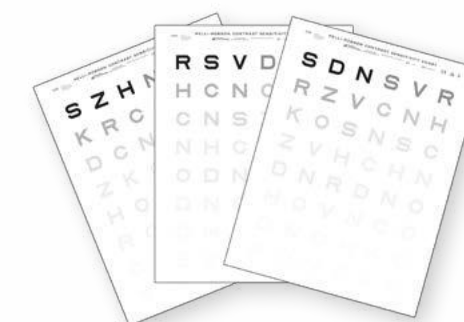

Pelli-Robson Near Contrast Sensitivity Charts

Set of 3 Sloan charts with mixed optotypes to avoid memorisation. The charts should be wall mounted at 40cm testing distance. Includes a durable case for safe storage.

Order code: **PV5013**

Pelli-Robson Trans-Illuminated Near Contrast Sensitivity Chart

Single Sloan letter chart for use in a small LED illuminator cabinet (26.5cm x 36.5cm x 3cm).

Order code: **PV5013-Trans**



Our carefully designed range of occluding glasses and patches make uniocular vision testing quick, easy and fun at all ages.



Occluding Patches with Wallet

There are three occluding patches 6.5cm across, plus one 5cm black patch for additional black out and to support the patch attaching to smaller size lenses.

Order codes:

Infant (Smiley, Butterfly, Skull, Plain Black) **OPCHI**

Junior (Camouflage, Hearts and Flowers, Stars, Plain Black) **OPCHJ**

Adult (4 x Plain colours) **OPCHA**



Infant



Junior



Adult



Large Occluding Glasses

Large glasses with one occluded lens for vision testing. Set of two pairs, navy and brown colour. They measure 14.5cm (5.7 inches) across the inside of the frame. Age range approx 8 years-adult.

Order code: **GLSL**



Regular Occluding Glasses

Robust plastic glasses for uniocular vision testing. Set of two pairs, neon orange and green in colour. They measure 11.5cm (4.5 inches) across the inside of the frame. Age range approx 4-8 years.

Order code: **GLSR**



Junior Occluding Glasses

Robust plastic glasses for uniocular vision testing. Comprises one pair with sunflower flip-up lenses to occlude each eye in turn.

They measure 10.5cm (4.1 inches) across the inside of the frame. Age range approx 3-5 years.

Order code: **GLSJ**



Infant Occluding Glasses

Robust plastic glasses for uniocular vision testing. Set of two pairs.

They measure 10cm (4 inches) across the inside of the frame. Age range approx 3 years or younger.

Order code: **GLSI**



Heart Occluder

Our Heart Occluder has been designed as a cheerful, child-friendly tool for many uniocular clinical assessments.

The curved shape fits and occludes the eye perfectly. The handle is easy to manipulate for cover test repeat movements, and long enough to hold away from the face when required.

Order code: **OCCLHEART**



Spielmann Translucent Occluder

This occluder provides total form occlusion during the cover test yet allows the observer to see the position of the eye behind the occluder. During vision testing, this opaque occluder can be useful in achieving the best visual acuity from someone with manifest or latent nystagmus.

Metric and inch measurements are provided on one side of the handle, pupil gauge on the other.

Order code: **SPIELTRANS**



Romanes Occluder

Used for the assessment of visual acuity and the cover test. One side of the handle has a millimetre scale for inter-pupillary distance measurement and for numerical measurement of ptosis. The other side incorporates a white dot for use as a scotometry target or fixation point.

Order code: **OCCLROM**



Lorgnette Occluder

Multi-purpose occluder that can be used to cover either eye. It is also ideal to quickly introduce a multi-hole pinhole to the uncovered eye.

The multiple pinhole attachment has a cluster of 15 holes. It pivots down in front of the eye to allow quick access to a pinhole view.

Order code: **OCCLORGN**

Occluders

These hand-held occluders are made of tough plastic that will not break or shatter. They are moulded into a smooth, rounded shape to fit comfortably over the eye, with no sharp edges. Their cups measure 6cm in diameter each and the handles are 15cm in length.



Black occluder with long handle.

Order code: **OCCLBL**



Multi pinhole occluder with long handle

Order code: **OCCLPIN**



Adjustable Flipper Lenses

Binocular, spherical plus/minus lenses with a sliding scale to adapt to a person's inter-pupillary distance. The frame will adjust between 4.25cm and 8.5cm to match a large range of inter-pupillary distances.

Order codes:

±1.00 DS: **FL1** ±1.50 DS: **FL1.5**

±2.00 DS: **FL2** ±3.00 DS: **FL3**

Near Point Dot/Letter Cards

Double-sided cards for measuring the near point of convergence (dots) and of accommodation (letters), and for home exercises. Full user and patient instructions included.

Pack of 50.

Order code: **NPD50**




Dinosaur Vision Therapy Cards

These transparent plastic cards have been designed to help with home training of monocular and binocular vision skills.

The multi-purpose card has three grades of stereogram and exercises for convergence, jump versions, focusing, fixation, saccades and directionality.

The cards are available in packs of ten with one comprehensive instruction booklet included.

Order code: **DC10**

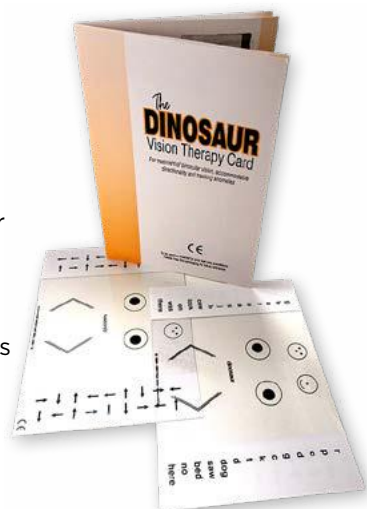

Flipper Lenses

Plus and minus flipper lenses in five standard sizes used for the diagnosis and treatment of accommodative defects.

Order codes:

± 1.00 DS: **FL1A** ±2.00 DS: **FL1C**

±1.50 DS: **FL1B** ±3.00 DS: **FL1E**


+2.00/-4.00 DS: **FL1D**

Other sizes may be available on request.

Unglazed Flippers

Pack of 10 plastic flipper frames in assorted colours.

Order code: **FLU**

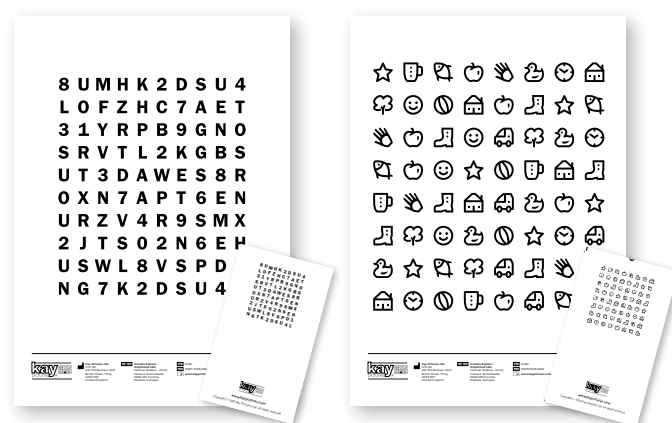

Key Accommodative Rock Charts

Used as an exercise to improve accommodative flexibility and accuracy. Also of use in treating those with specific literacy difficulties. Two sorts, one with letters/numbers and one with pictures. Comprises laminated near and distance cards and full instructions for use. Available in packs of 5 or 50.

Order codes:

Pack of 5

Letters/numbers: **KARL**

Pictures: **KARP**

10 Packs of 5 each

Letters/numbers: **KARLX10**

Pictures: **KARPX10**

Brock String

Three coloured beads on a 2-metre length of polyester (non-stretchy) cord for home exercises aimed at improving binocular alignment.

Each bead is 10mm in diameter.

Exercises involve positioning the three beads at different distances along the cord and moving the eyes to look at or follow the beads.

Basic patient instructions are included. Available in packs of 10.

Order code: **BROCK**



Hess Screen

Hess Screen is used to test the function and co-ordination of the extra-ocular muscles to aid the diagnosis of ocular motility disorders.

The full set comprises a 92 cm x 92 cm size electronic grey screen with 25 red lights, a pack of corresponding recording charts, a green laser pointer with AAA batteries, red/green dissociative goggles and a power unit.

Order Codes:

Hess Set: **HESS-331-SET**

Chin Rest: **HESS-331-CRA**

Laser Pointer: **HESS-331-LP**

Recording Charts: **HESS-313-RC**

Goggles: **HESSGOGGLES**



Stick Prisms

Individual stick prism mounted on a metal handle for easier use. Holding the stick, rather than the prism itself, keeps the prism free of finger marks and your hand away from the face when testing.

Four sizes available: 4, 10, 15 and 20 Diopter.


Order codes:

4D: **PRISM4D**

10D: **PRISM10D**

15D: **PRISM15D**

20D: **PRISM20D**

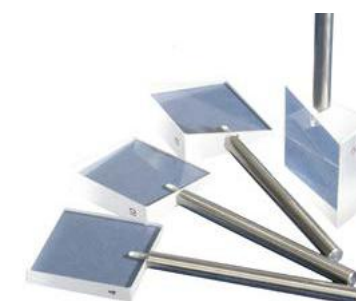

Dominant Eye Test Box

Simplifies the process of determining the patient's dominant eye. Very quick and easy for children to understand. Made from wipe-clean plastic. Provided in easy to assemble flat pack with full testing instructions.

Order code: **DET**



Frisby Stereo Tests

Measure a wide range of stereo acuity in true 3D without the need for dissociating glasses. Infants as young as 8 months can understand and comply with looking or reaching for the target. Stereo vision can be diagnosed even when acuity is poor.



Complete 3 Plate Test
Order codes:
Complete Test: **FRISP**
Replacement Box: **FRISBOX**




Pocket Test
Order codes:
6mm plate: **FRISP**
6mm & 3mm plate: **FRISPS2**

TNO Stereo Test

Used for stereopsis testing in occupational environments. It consists of three screening plates, a plate for testing eye suppression and two plates for the quantitative testing of stereo acuity. Available with stereo viewing glasses for either children or adults.

Order codes:
Book with Child Glasses: **TNOSETCHILD**
Child Glasses: **GOGGLESCHILD**
Book with Adult Glasses: **TNOSETADULT**
Adult Glasses: **GOGGLESADULT**
Book: **TNOSTEREO**

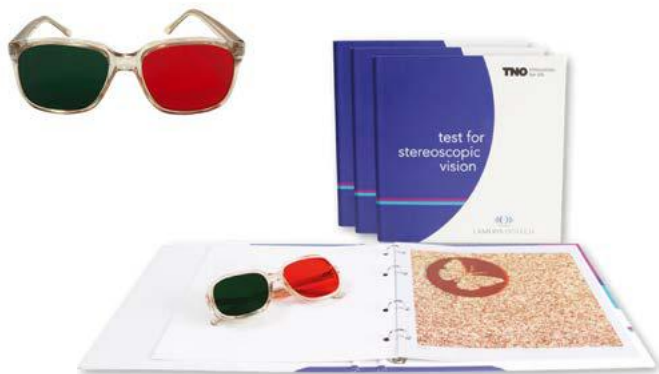

Lang Stereo Tests

Measures stereo acuity easily in children from 30 months and adults without the need for dissociating glasses. Three levels of stereo acuity on one handy card. Two options to test different levels of stereo acuity.

Measures from 1200 to 400 seconds of arc
Order code: **LANG1-R**


Measures from 600 to 200 seconds of arc
Order code: **LANG2**



Lang Stereopad

Double-sided A5 test panel with magnetic test cards for the assessment of stereopsis. The test does not require specific glasses and is highly sensitive for global stereopsis. It has no monocular clues and is ideal for preferential looking testing of infants and preverbal children.

Order code: **LANGPAD**

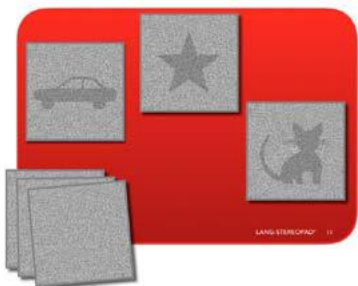

Kay Fun Patch is our trusted brand of high-quality, medical fabric eye patches made entirely in the UK

- Soft, cushioned & breathable
- Certified & hypoallergenic materials
- Reusable & hand washable
- Individually hand-sewn
- Cost effective
- Environmentally friendly



INNER LAYER
100% COTTON

Kay Fun Patches

Fabric occlusion patches fitting most sizes and styles of glasses and available in a wonderful range of colourful designs. Packs of 10, 50 or more. Assorted designs, all boys, all girls.

Order codes:
10 Assorted: **KFPCH10E** 50 Assorted: **KFPCH50E**
10 Girls: **KFPCH10EG** 50 Girls: **KFPCH50EG**
10 Boys: **KFPCH10EB** 50 Boys: **KFPCH50EB**



Baby Fun Patches

Smaller-sized fabric patches with additional elastic fixation on the glasses arm, especially designed for infants. They measure 10.5cm x 5cm and are available in packs of 10 assorted plain colours or packs of 10 assorted patterned designs.

Plain Baby Fun Patches

Order code: **BFPCH10PL**

Patterned Baby Fun Patches

Order code: **BFPCH10**



Plain Fun Patches

These plain coloured patches are more economical to buy yet made with the same care and attention to quality as all our patches. Pack of 10 assorted colours including coral, pale pink, navy, almond and sand.

Order code:

KFPCH10EPL



Pirate Fun Patches

Black fabric patches with red or blue sparkling pirate skull and crossbones motifs or neon pink and glow in the dark. Packs of 10 each (5 right, 5 left).



Glow in the Dark

Order code:
PFPCH10GLOW



Red Sparkle

Order code:
PFPCH10REDSP



Blue Sparkle

Order code:
PFPCH10BLUSP



Neon Pink

Order code:
PFPCH10NEOP



**NEW
FUN PATCH
DESIGNS**

Junior Eye Patches

Eye patches for children without glasses, made entirely from reusable, washable fabric. Suitable to wear over the right or left eye. Assorted, colourful designs for girls and boys.

Order codes:

10 pack girls designs: **EPCH10JG**

10 pack boys designs: **EPCH10JB**

10 pack assorted designs: **EPCH10J**

50 pack assorted designs: **EPCH50J**



SIZES

R: 13cm x 6cm
L: 14cm x 6cm

Adult Glasses Patches

Adult glasses patches with elastic, suitable for all spectacle frames, with or without separate nose pieces, and available in two sizes in assorted, discrete colours.

Order codes:

Regular 10 pack: **AGPCHR10E**

Large 10 pack: **AGPCHL10E**

Regular 50 pack: **AGPCHR50E**

Large 50 pack: **AGPCHL50E**



Plain Black Adult Glasses Patches

Pack of 10 plain black adult glasses patches, available in regular and large.

Order codes:

Regular: **AGPCHR10BLK**

Large: **AGPCHL10BLK**



Patterned Adult Glasses Patches

Pack of 10 assorted adult glasses patches, only available in regular.

Order code: **AGPCHR10PAT**



**NEW
PATTERNED
GLASSES
PATCHES**

NEW EYE PATCH DESIGNS



Patterned Adult Eye Patches

Pack of 10 assorted, regular-sized adult eye patches with stylish patterns. Packs are ordered as a set of 10 assorted men's designs or 10 assorted women's designs.

Order codes:

Men's: **EPCH10APATM**

Women's: **EPCH10APATF**



Adult Eye Patches

Eye patches for adults without glasses, made entirely from reusable, washable fabric. The patches are held comfortably in place with 7mm wide and 40cm long quality, stretchy elastic, which is also latex free. Available in a small and regular size.

Order codes:

Regular

10 pack assorted colours: **EPCH10A**

50 pack assorted colours: **EPCH50A**

Small

10 pack assorted colours **EPCH10ASM**

SIZES

S: 7cm x 5.4cm
R: 7.7cm x 6.7cm



Kay Magic Patches

These inconspicuous, adhesive patches for glasses obscure all form to the wearer yet leave the eye behind visible. They are suitable for amblyopia and intractable diplopia treatment and are easily removed from the lens. 12 patches are provided in a rigid, vinyl wallet. Replacement refill packs of 12 patches are also available. Each patch is 8cm x 6.5cm in size. Pack contains 12 large patches.

Order code:

12 patches in a vinyl wallet: **KMPCHWAL**

Refill: **KMPCH**



Animated Pictures Stick

Double-sided plastic fixation sticks ideal for keeping a child's attention during assessments.

Each stick has animated pictures on one side and reduced LogMAR letters on the reverse. FS1B and FS1C also show reduced LogMAR size Kay optotypes to help maintain constant accommodation during testing.

Order codes:

Winking Eye: **FS1A**

Monkey / Frog: **FS1B**

Monster / Butterfly: **FS1C**

Infant Fixation Sticks

Double-sided plastic fixation sticks ideal for keeping a child's attention during assessments.

Infant designs. Bold colourful pictures with lots of detail, plus letters and Kay Picture optotypes.

Order code: **FS3A**



Junior Fixation Sticks

Double-sided plastic fixation sticks ideal for keeping a child's attention during assessments.

Junior designs. Bold colourful pictures with lots of detail. Underneath the pictures are rows of N5 size upper- and lower-case letters to read, plus one row of N8 size.

Order code: **FS3B**



Lang Fixation Cube

The Lang Fixation Cubes have a different picture on each of the five sides of the cube. The simple pictures are bold and colourful, ideal to attract the attention of young children during vision testing.

Order code: **CUBER** Order code: **CUBER**



Plain Fixation Sticks Pack

Pack of 5 plain white PVC plastic sticks to add your own fixation targets.

Wipe clean and reusable.

Order code: **FS5PLAIN**

Pen Torches

Medical pen torches and attachable finger monsters in assorted colours.

Replaceable batteries.

Available individually or in packs of five.

Order codes:

Single: **PENT1M** Pack of 5: **PENT5M**



Mini Disco Ball

Hand-held, array of flashing, coloured lights with three speed settings at the press of a button. The changing, colourful patterns provide a silent, interesting fixation target. Yellow or blue with three AAA batteries included.

Order codes:

Single: **DISCO**



Spinning Lights

Hand-held colourful spinning lights with changing pattern to maintain fixation and interest. Useful when testing infants and young children.

Three AAA batteries included.

Order codes:

Single: **SPIN**



Assorted Torches

Choose from various packs of three assorted torches. Most sets also have a sound effect. They are great to attract the attention of young children during testing. The light will reveal corneal reflections and the sound will keep the child interested for tests such as ocular motility. Button batteries included.



TA3B



TA3C



TA3A (no sound)



TA3D

Jungle Charts for Patches and Glasses

These charts help children remember to wear their patch or glasses and take pride in their success. Drawing a smile on a different animal each day gives the child a visible, positive record of their ongoing achievement over four weeks. Size: A4.



Order codes:

Jungle Patch Chart

100 pack: **JPC100**

500 pack: **JPC500**

Jungle Glasses Chart

100 pack: **JGC100**

500 pack: **JGC500**



Amblyopia Activity Book: 'Hoppity Frog Has Two Clever Eyes'

Story, colouring and activity book designed for children who are having treatment for amblyopia. The rhyming story is an account of having an eye test and getting glasses and an eye patch, designed to help children to understand their patching treatment in a simple, fun way.

Order codes: 10 pack: **HOPP10**

50 pack: **HOPP50**



Patching Calendar

Four-week calendar and sticker packs to help with patching. The calendar has windows to open that reveal a picture and positive message. Encourages patching by association with adventurous, difficult tasks, like climbing a mountain.

Order codes: 10 pack: **CALEN10**

50 pack: **CALEN50**



Toy Glasses

Realistic toy glasses in pink and blue to aid glasses compliance through play. Packs of 10 or 50 (equal mix of blue and pink).

Order codes:

10 pack: **TOYGLS10**

50 pack: **TOYGLS50**



Jungle Certificate

Reward a child for their effort and achievement with these fabulous certificates. Simply insert the child's name and add what big or small success they've achieved. Pack of 100 or 500.

Order codes: 100 pack: **JCERT100**

500 pack: **JCERT500**



Reward Stickers

Large, 42mm diameter, shiny stickers will please and reward any child. Use as an incentive for the child to complete a task, or as a simple reward.

Order codes for rolls of 500 stickers with the same design: **S500 (+ ref code)**



STP



BRL



FAN



AWA



BEY



STA



EYE



YHO



BEE

Assorted Stickers

Sheets of 24 colourful reward stickers with many different pictures and messages to suit any child. They are 40cm in diameter and come in a pack of 20 sheets (480 stickers).



Bravery and Behaviour Stickers

Order code: Pack of 480: **SB480**



Optician and Eye Stickers

Order code: Pack of 480: **SO480**



Orthoptist and Eye Stickers

Order code: Pack of 480: **SE480**

Fun Picture Search

Our **FREE** Fun Picture Searches are available as a resource to give to your patients undergoing patching treatment. Visit www.kaypictures.co.uk/fun-picture-search



FREE POSTERS

Brighten up your practice, waiting room or department with our selection of informative, colourful, and engaging posters.

Contact us for printed A3 copies or visit our resources page for FREE, printable, A4-sized downloads.



Kay Say & Match


The free* interactive app for young children to learn the Kay vision test pictures at home.

Get ready for your eye test with my fun and easy games!

Available on the App Store and Google Play.

kay pictures  **35 years** www.kaypictures.com

*in-app purchase for a speaking matching card
Copyright © 2022 Kay Pictures Ltd

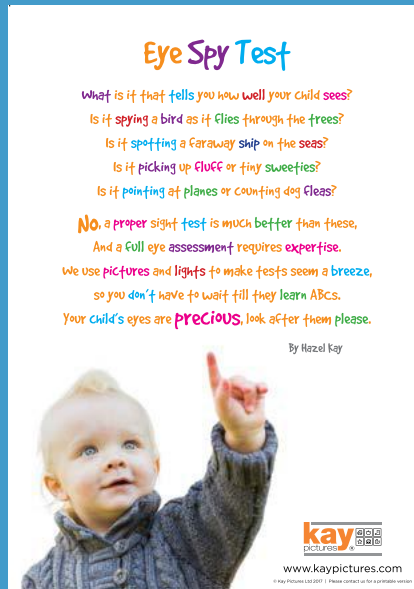


Eye Spy Test


What is it that tells you how well your child sees?
Is it spying a bird as it flies through the trees?
Is it spotting a faraway ship on the seas?
Is it picking up fluffy or tiny sweeties?
Is it pointing at planes or counting dog fleas?


No, a proper sight test is much better than these,
And a full eye assessment requires expertise.
We use pictures and lights to make tests seem a breeze,
so you don't have to wait till they learn ABCs.
Your child's eyes are precious, look after them please.

By Hazel Kay

kay pictures  www.kaypictures.com

© Kay Pictures Ltd 2022. All Rights Reserved. For a complete list of our products visit www.kaypictures.com

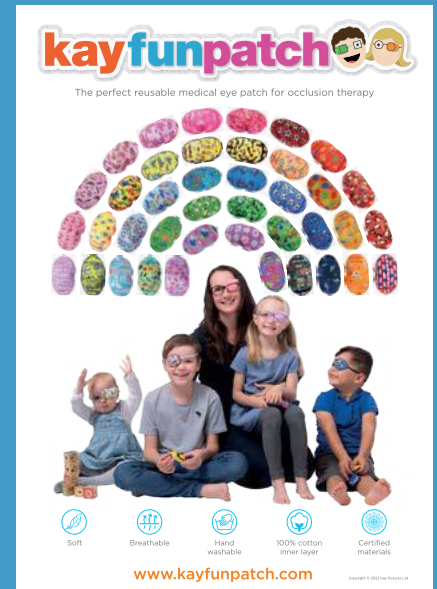


kayfunpatch 

The perfect reusable medical eye patch for occlusion therapy

Soft, Breathable, Hand washable, 100% cotton inner layer, Certified materials


www.kayfunpatch.com

Copyright © 2022 Kay Pictures Ltd

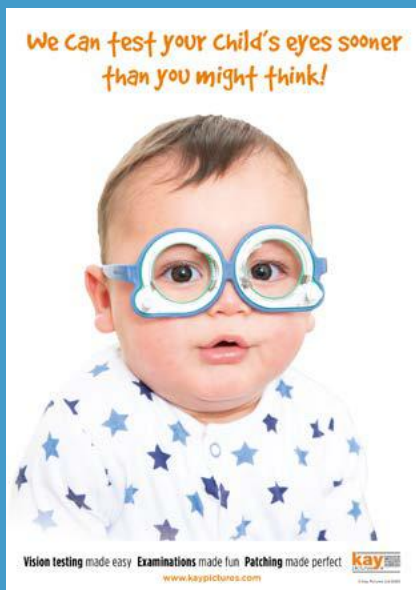

We can test your child's eyes sooner than you might think!

Vision testing made easy. Examinations made fun. Patching made perfect.

kay pictures  www.kaypictures.com

© Kay Pictures Ltd 2022



Say & Match

The free* interactive app for familiarisation with the Kay Picture optotypes

A fun and interactive way for very young children to learn the Kay Picture optotypes prior to an eye test. For children from 15 months of age who are just learning to talk, and to help children over 24 months learn how to match.

Download from the App Store, Google Play or via the link on our website at www.kaypictures.com

kay pictures  www.kaypictures.com
[@kaypictures](https://twitter.com/kaypictures)
[kaypictureshd](https://www.facebook.com/kaypictures)

*in-app purchase for a speaking matching card

Order this pack of 50 free* leaflets with useful information about the Kay Say & Match app to help your patients prepare for their eye test.

*Shipping fees apply if not part of an order.

For our full product range and further information please visit:
www.kaypictures.com and www.kayfunpatch.com



Kay Pictures Ltd
Unit 39, Silk Mill Business Park
Brook Street, Tring, Hertfordshire HP23 5EF
T +44 (0) 1442 823507
E contact@kaypictures.com
www.kaypictures.com 